




Refer to User Manual for comprehensive information:  
<http://www.mttts-asia.com/firefly-phototherapy/>

## Control Panel

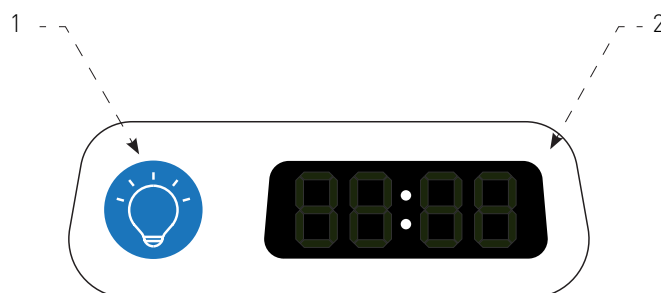

1. Power button - Turns the device ON and OFF.
2. Display - Shows the 'treatment time', as well as 'total device hours' of Firefly usage.

## Warnings



**Error messages:** In the case that the Firefly has a minor error then the error number will be displayed on the control panel. The Firefly is still safe to operate, however please contact MTTTS and quote the error number and device serial number.



**LED lifetime warning:** The blue LEDs in the Firefly are rated to provide effective treatment for 60,000 hours.

## Safety




DO NOT carry the Firefly while the patient is in the device.



When using Firefly phototherapy the patient should ALWAYS be placed in the bassinet.

## Irradiance

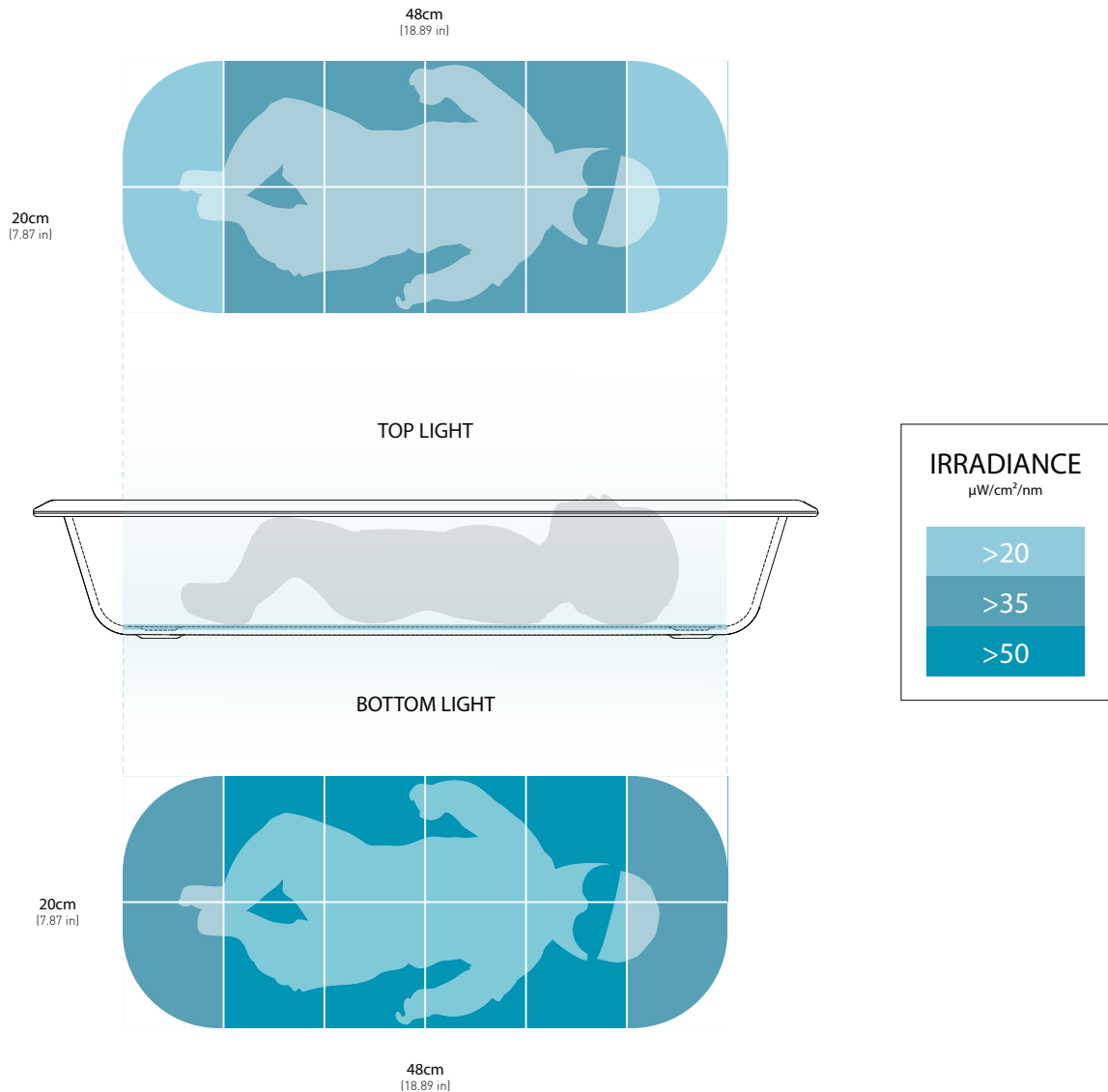

## Cleaning and disinfection



- ✓ **Only disinfect the bassinet** with a solution containing at least 70% alcohol. Apply the disinfectant with a clean cloth or soft sponge. Wait for the device to dry after cleaning or disinfecting before use.
- ✗ DO NOT use alcohol to disinfect any other part of the device.
- ✗ DO NOT immerse any part of the device in water or alcohol.